



AMAL
GLASS

Summary of Technical Specifications and Features

For - Amal Glass





Features of Amal Glass

What is the Amal glasses for the blind and what distinguishes them?



AMAL GLASS is a smart glasses that help the blind to be independent through many features that help the blind to continue their daily activities without anyone interfering to help.

It is a smart glasses based on artificial intelligence and similar to sunglasses and contains a camera and sensors, and it works through a mobile application, the Android system.




What is the Amal glasses for the blind and what distinguishes them?



Spectacles are a main need of the blind, so that they help them in their daily lives.

The product supports the full Arabic language for all services, and this is not available in the assistive devices for the blind and visually impaired from foreign companies, which facilitates the use of the Arab blind for the product.



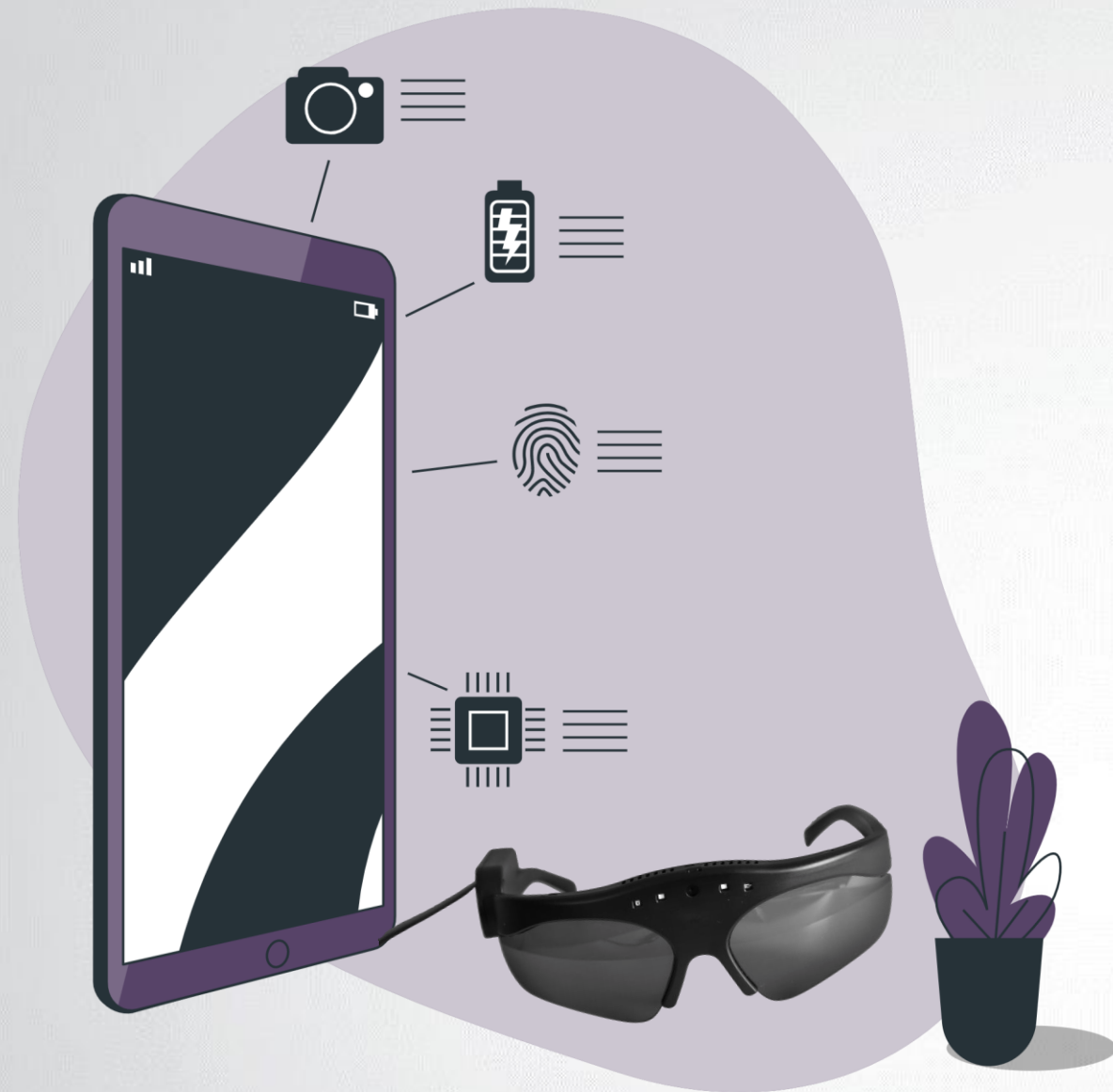

Amal Glass Apps

The applications that the glasses contain are:


Read barcode and QR code



Reading written texts, computer screens, mobile devices, and paintings in more than one language, and recording the reading for later reference.



Requesting assistance by voice calling and geolocation when requesting assistance from the numbers that the blind person has registered on the system previously.

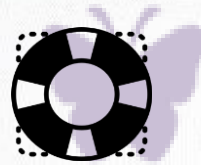

Get to know things. in the surrounding environment.



Specify the address of the place where the user is located.



Learn about banknotes of all kinds.



The applications that the glasses contain are:

Warning in dark areas.



The weather in your location.



Identify the lighting in the place.



Speaking clock and setting the time, day, Hijri and Gregorian date and stopwatch.



Record voice notes.



Determine the direction of the qiblah.



The applications that the glasses contain are:

Adjust language and reading speed



Play audio notes.




The translator



The Flashlight



Adjust the sound

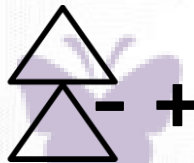

Remote control.



The functions of the glasses clearly distinguish them, as they are superior to other global products, as evidenced by the comparison with the most important competitors

Glasses have nothing to do with medicine from a clinical or preventive point of view, but only from the psychological and living point of view of the patient. They are necessary in social and practical terms in improving the life of the blind or visually impaired and raising the quality of life for this category. They do not replace the blind with the use of the white stick for inference during movement.



Instructions for using the glasses

Instructions for use



01

Ensuring that the person concerned needs it and benefits from the functions supported by AI-Amal glasses that are shown in the attached document before the purchase decision, knowing that the current version of the glasses does not contain anything other than what was mentioned in the document.

02

The price without a mobile phone is 1100 USD or its equivalent in other currencies, including the delivery values for the customer's country. With a Samsung mobile, an amount equivalent to 1400 USD, plus sales tax or customs, according to the buyer's country.

03

You can buy or use other types of mobile phones of the highest category, such as Samsung A71 or others that run on Android.

04

The glasses do not work on an iPhone mobile phone, but rather work on an Android mobile application, for example Samsung, as mentioned above.

05

The transfer is made before the glasses are shipped. When transferring, please mention the data in the transfer form in full, including the reason for the transfer, which is the purchase of blind glasses, and the amount is net of the transfer expenses.



Instructions for use



06

Please send a copy of the transfer on WhatsApp to the number +966555425566 with the name of the transferee, mentioning the name of the beneficiary of the glasses, and clarifying the address of who will receive the glasses in full in the country and city.

07

In addition to the transfer amount, tax is added, for example, if the buyer in Saudi Arabia is 15%, and in the United Arab Emirates 5%. Otherwise, the recipient pays it in his country and according to the system of the country in which the customer is located.

08

What the user needs to know how to operate the glasses' mobile and it has nothing to do with the user's mobile is to operate the glasses only and does not require any skill from the user, only activates it and connects the glasses while it recognizes the program and starts working on its own.

09

There are no fees for the programs and services in the glasses for the first year, as they are free for the customer, and after the end of the first year there is an annual fee of 200 UAE dirhams or its equivalent in other currencies for paid services and programs, or the equivalent of \$5 per month as a maximum.



Instructions for use



09

The warranty is for a period of one year from the date of purchase against manufacturing defects, and the device is replaced directly, but in case of damage from the user, the device is replaced and the customer pays the amount of damage and gets another device.

10

The customer must have a chip or Wi-Fi internet service, because the glasses need internet for some important applications.

11

Any inquiry should be sent to the company's email info@amalglass.com < or on WhatsApp 966562052050.



Thank you for your support and interest